FLOCARE® NASOGASTRIC FEEDING TUBES

EN • Patient Information – FloCare® PUR, PURSOFT and NUTRISOFT feeding tubes

1. What are FloCare feeding tubes and what are they used for?

Nasogastric feeding tubes are intended for delivery of enteral nutrition in the stomach.


2. Warnings and precautions:

For enteral use only; choking hazard/ loose parts; single patient use only; do not re-use once removed as this may create a risk of infection and cause the product not to function as it should. Consult your healthcare professional before using the device or in case of any doubt. Any serious incident that has occurred in relation to this device is to be reported to Nutricia and the local competent authority.

3. How is the product used?

Connect the tube to a giving set or syringe with ENFit connector and begin feeding per healthcare professional's instructions.

Daily care: Vomiting, retching, coughing or accidentally pulling the tube could displace the tube. Should this happen, fluids could go into the lungs instead of the stomach. For this reason, always check the position of the tube via the method indicated by the managing healthcare professional. Contact your healthcare professional in case of doubt and never try to reinsert your feeding tube. Keep the tube free from (feed) debris and clean the connector regularly. Flush the tube with water using an enteral syringe of at least 20mL, before and after the administration of feed or medication and at least every 8 hours to prevent blockage of the tube. Never use excessive force, if there is any resistance, stop the procedure and consult your healthcare professional.

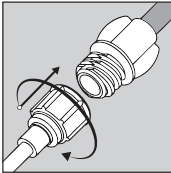4. What are the possible complications?

Tube blockage, irritation around the nostrils caused by the tube, nausea, vomiting and aspiration can occur. Consult your healthcare professional in case of any observed complication.

5. How to remove and dispose the tube?

It is recommended to use the FloCare® nasogastric feeding tube for <6 weeks to minimize the risk of complications caused by material degradation or clogging within the tube. Removal should always be performed under medical supervision.

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